BeyondSOL

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Contents

1. Safety Instruction
2. Component
3. How to Use
4. Specifications and Characteristics
5. Application

Safety Instruction

Please read the 'Safety Instruction' and 'How to use' carefully before use.

Wireless Device

This wireless equipment may cause radio interference during operation. Therefore, we can not provide services related to personal safety. Before boarding any aircraft or packing a wireless device in the luggage that will be checked, turn the wireless device off.

Earbud Device

Extended exposure to high volumes when using the earbud may result in temporary or permanent hearing loss. To prevent damage to your earbuds, avoid sitting or stepping on it.

Battery-Powered Device

Improper use of batteries may result in battery fluid leakage, overheating, or explosion. Do not place the product near or in a hot source. It may cause explosion and fire.

Do not allow metal objects to touch the battery terminals on the device, they can become hot and cause inflammation.

Safety Instruction

Please familiarize yourself with the instructions.

Check your ears for massive earwax or foreign objects.

If there is any foreign substance on the product, remove the foreign substance.

Safety Instruction

Please read the 'Safety Instruction' and 'How to use' carefully before use.

Caution

1. To reduce the risk of fire or electric shock, do not expose this device to rain or moisture.
2. Do not attempt to take apart, open, service, or modify the device, doing so could present the risk of electric shock or other hazard.
3. Do not use this device in a place exposed to high temperature, high humidity or dust. Do not wear this product when receiving medical treatment or radiation treatment at a medical institution.
4. If water or sweat enters the device, it may damage the internal parts and cause malfunction. If water gets in to the device, wipe it with a dry cloth immediately and contact our service center.
5. When decorating or coloring the product arbitrarily, the appearance and surface of the product may deteriorate and cause an irritation depending on the constitution. If this reaction occurs, discontinue use and consult a physician.
6. If you want to control the sound through the application, please familiarize with how to use the app.
7. Excessive high volume may cause temporary or permanent hearing loss.
8. Keep the product clean. Use in dusty environments may cause product failure.
9. Keep earbuds and eartips as clean and dry as possible.

Safety Instruction

Please read the 'Safety Instruction' and 'How to use' carefully before use.


Caution

10. To conserve battery power, turn off the product when not in use.
11. If there is a change in sound quality, remove the ear tip and check the earbud nozzle. If you have any problems, please contact our service center.
12. Lithium-ion battery can be replaced by authorized person of manufacturer using tool. Replace of battery by non-authorized personnel can cause the risk to operator.
13. Interference many occur during treatment to other devices, it has a potential risk possibility. Therefore, please operate the device at the environment recommended by manufacturer.
14. There is possibility to affect the EMC performance if operator is not used parts that is recommended by manufacturer. And if the damage by using unspecified parts, manufacturer does not warranty for damage of the device and parts.
15. If the product is dropped from a high place, it may cause damage to the product so handle with care.

Component

Product

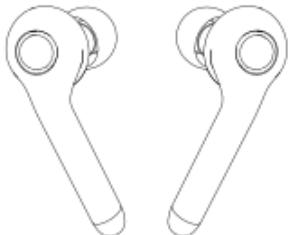Earbuds


Cradle

Extra Eartip

Small

Charging Cable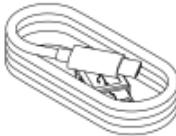

Quick Manual

How to Use

Power On/Off

When the Earbud is removed from the Cradle, the power is turned on.

When the Earbud is inserted into the Cradle and lid is closed, the power is turned off.

* When opening the package box for the first time, put the Earbud into the Cradle, close the lid and open to turn the power on.

* If the battery in the cradle is discharged, normal power on / off operation may not be performed. In this case, Please charge the cradle before use.

You can also turn the power off by pressing the button on the earbud for 5 seconds, and turn the power on by pressing the button when it is turned off.

* RED LED (Charging) → BLUE LED (Charged) will be displayed when the earbud is properly placed in the charging case.

* If the BLUE LED blinks immediately when the earbud is charging, the charging pin is not properly connected. In this case, place the earbud on the cradle again so it is properly placed. Check the RED LED (Charging) → BLUE LED (Charged).


How to Use

Hearing Aid Mode

You can turn the hearing aid mode on and off by pressing the button on both earbuds for longer than one second.

- * The hearing aid mode does not turn on automatically when the earbud is turned on. Press and hold the button for 1 second to turn on the hearing aid mode.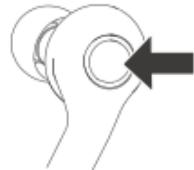

- * If you play music on your Smartphone while in hearing aid mode, the music will be played on your Smartphone speaker. If you want to listen to music with earbuds, Please exit the hearing aid mode by pressing and holding the button for 1 second to change to Bluetooth mode.

How to Use

Bluetooth Connectivity

- 1) Put the Earbud in the Cradle, Close and reopen the lid to enter the Bluetooth pairing mode.
 - 2) Search for Bluetooth setting in your phone and select EM-V300_L.
 - 3) At this time, click 'OK (Pair)' when pop-up shows to connect additional EM-V300_R.
 - 4) The smartphone and EM-V300_L are connected by Bluetooth, and EM-V300_R is registered additionally.(The EM-V300_L is connected to the Bluetooth screen of the smartphone.)
- * Do not remove both earbuds from the cradle until all the above steps are completed when connecting the Bluetooth.
 - * If you want to change the main unit of the earbud (device connected to the smartphone) You can change the main unit by selecting the current connected device (Ex, EM-V300_L) from the smartphone and disconnecting it, then selecting the desired device (Ex, EM-V1300_R).
 - * While using a specific Media Player to listen to music and main unit is changed or hearing aid mode is turned on or off, the music will be played on the smartphone speakers.
 - * When connecting for the first time, the EM-V300_L will be connected to the main unit. Then later on, when connecting the smartphone again, the earbud with a lot of battery power is connected to the main unit for efficient use of the product.
 - * Main use device: The device on which the microphone is activated during a call

Caution

- * The wireless equipment may cause radio interference during operation, and Intermittent sound discontinuity may occur in AP mixed areas.

How to Use

Phone Call

*Connect to Bluetooth on your smartphone first before use.

Answer Call	When a call comes in, you can press the button on the earbud to answer the call.
End Call	To hang up the call, you can press the button on the earbud again to end the call.

When the call mode is ended, the earbud is automatically switched to the previous (music listening / hearing aid) mode.

Music Streaming

*Connect to Bluetooth on your smartphone first before use.

Play	When the Bluetooth is connected, press the button on the earbud to play the music.
Pause	While music is being played, press the button of the earbud to pause the music.
Next track	Press the button on the right earbud twice in order to play the next track. * It can be executed only when both earbuds are worn.
Previous track	Press the button on the left earbud twice in order to play the previous track. * It can be executed only when both earbuds are worn.


How to Use

Charging

Caution

The left and right side of the Cradle charger is different. Please insert the earbuds properly when charging.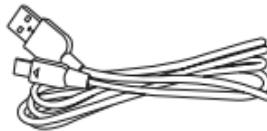

- If there is no battery in the earbuds, hearing aid mode will automatically turn off with a voice guidance. In this case, please charge the device before use.
- If the battery in the cradle is sufficient, insert the earbuds in their respective positions and close the cover to charge the earbuds.
- If the battery in the cradle is low, it may be possible to charge the earbud, so you can charge the battery by connecting the USB port on the back to the power source.

How to Use

1. Earbuds Charging

The LED colors of the cradle during charging are as follows

Battery Status	Earbud Charging	Earbud Charging Complete
LED	Red	Blue → Off

When the power is ON, the remaining battery level can be confirmed by voice.

2. Cradle Charging

LED display during cradle charging

Battery Status	Cradle Charging	Cradle Charging Complete
LED	Red	Blue

3. Cradle Battery Check During Carrying

You can check the remaining battery level by closing the lid while the lid is open while you carry the cradle.

Battery Status	Low	Medium	High
LED	Red	Purple	Blue

Specifications and Characteristics

Product Name	BeyondSOL	
Manufacture / Country	BSL Co., Ltd. / Republic of Korea	
Manufacture Address	Suite 104, GangwonTechno Park ChuncheonVenture 2nd Factory Sinbuk-ro 61-14 Shinbuk-eup Chuncheon-si Gangwon-do, Republic of Korea	
Operating Environment	10~40°C / 30~75%R.H / 70~106kPa	
Storage Environment	10~40°C / 30~85%R.H / 70~106kPa	
Usage Time	Hearing Aid Time	Max. 14 hrs*
	Streaming, Phone Call Time	Max. 3 hrs
Charging Time	Earbud 1.5 hrs / Cradle 2.5 hrs	
Bluetooth	Ver 5.0	
Acoustic Parameter	Channel / Band	5 CH / 32Band
	Driver Unit	Single BA
	Frequency Range	250Hz ~ 6,500Hz
	Sensitivity	107.5 ± 3dB SPL@1kHz
Power	Impedance	619 Ohm ± 15 % @1kHz
	Rated Input Voltage(Charging)	5V⎓ ; 500mA
	Cradle Battery	3.7V⎓ ; 400mA, Li-Polymer
	Earbud Battery	3.7V⎓ ; 32mA, Li-ion
BSL		


* The test was conducted by completely discharging the battery when entering Hearing aid Mode (Vol. 5). Battery life may vary depending on device settings, environment, usage, and many other factors.

Application

Download


BeyondSOL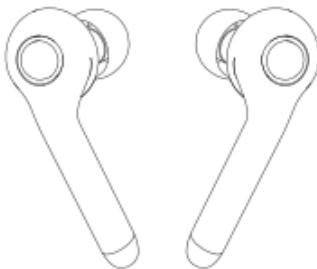


- 1) Android : Download on Play Store. iOS : Download on App Store.
- 2) Search for 'BeyondSOL' in the search box.
- 3) Select the BeyondSOL app distributed by BS^L and install it on your smartphone.

Application

Run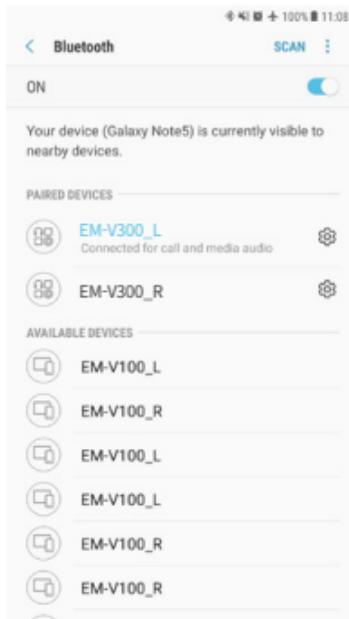


connect the product to the phone with Bluetooth to use the app.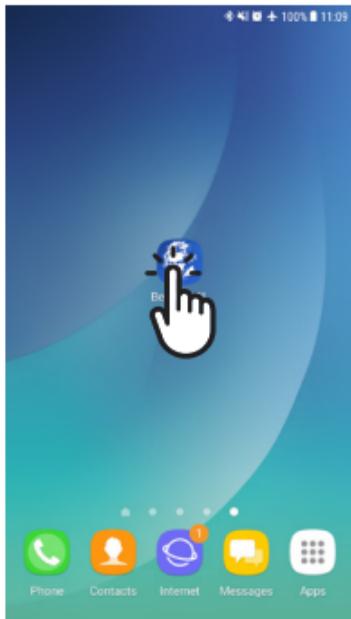


In the list of apps on your phone, find 'BeyondSOL' and select to launch the app

Application

Simple Fitting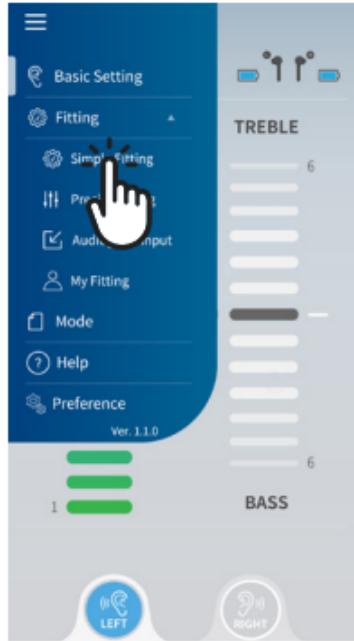



Select the menu (≡) in the upper left corner of the main screen and select 'Simple Fitting'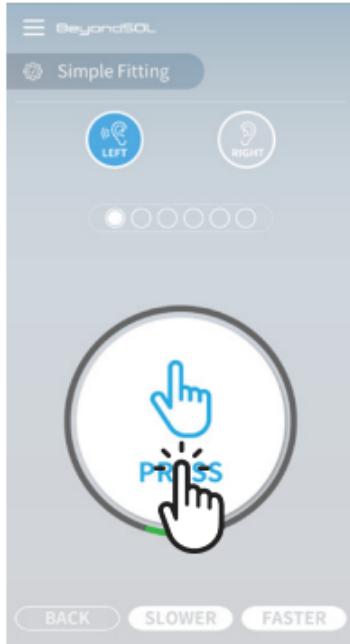

A beep sound will be played at a specific time interval, when you hear a 'beep' press the 'Press' button.

Application


Simple Fitting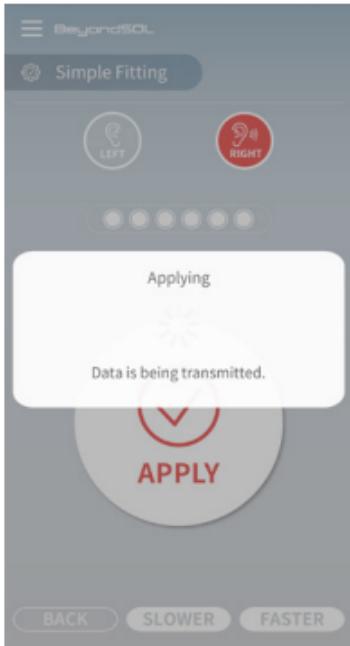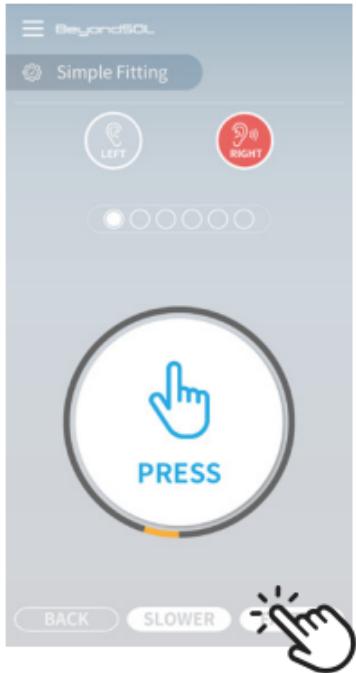


The test is performed left and right ear. When the test is done, the test results will be implemented to the product.

Application

Precise Fitting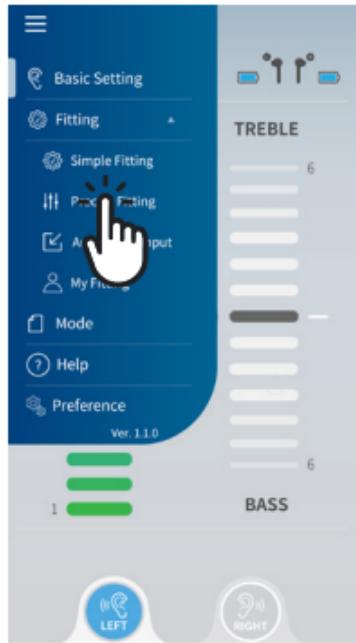

Select menu (≡) to open the slide menu and select 'Precise Fitting'

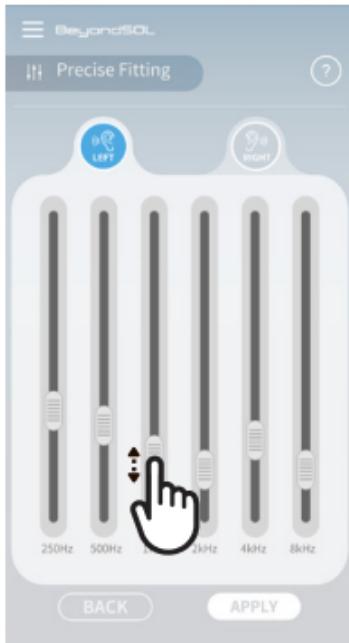
Move the six control bars to the lowest position where you hear the sound.

Application

Precise Fitting


Both Left ear and Right ear should be adjusted.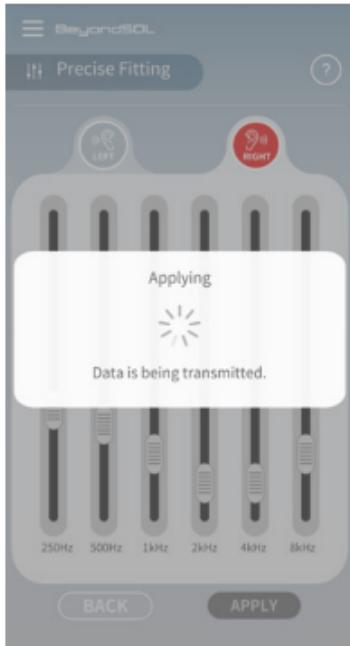

After the adjustment is completed, press 'Apply' and the test results will be implemented to the product.

Application

Audiogram Input

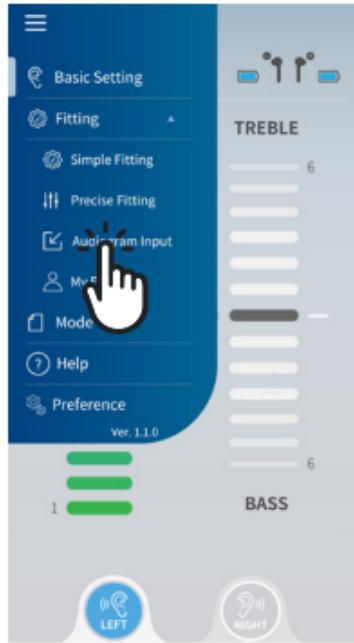
Select menu (≡) to open the slide menu and select 'Audiogram Input'


Scroll and center the number that matches your hearing and press 'APPLY'.

Application

Audiogram Input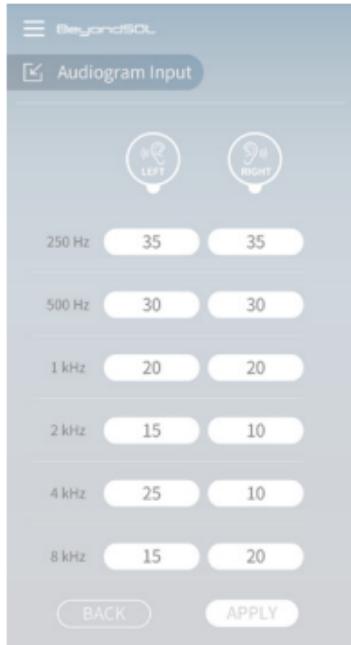


Both Left ear and Right ear
should be adjusted.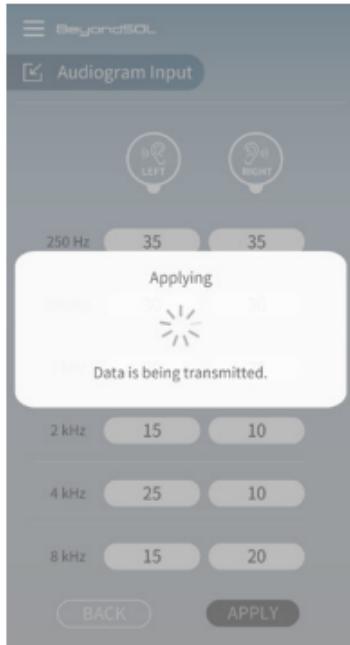


After the adjustment is
completed, press 'APPLY' and the
test results will be implemented to
the product.

Application

My Fitting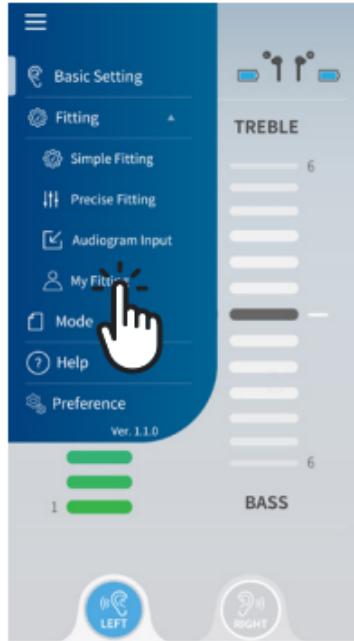


Select menu (≡) to open the slide menu and select 'My Fitting'.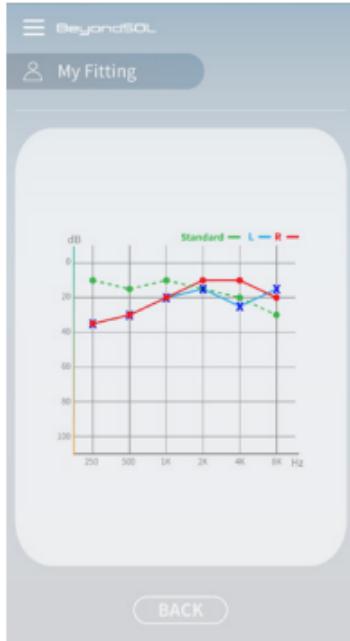

You can check the test result set in the current product.

Application

Mode

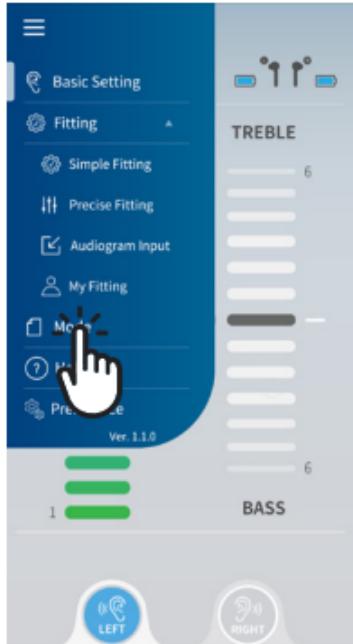
Select menu (≡) to open the slide menu and select 'Mode'. Listening mode can be selected automatic or manual


The 'ECS Mode' automatically sets the listening mode that is most suitable for your surroundings.

Application

Mode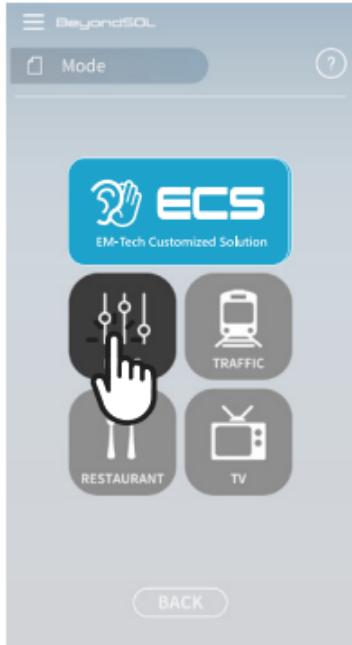


'Basic' mode is set up to be suitable for indoor / quiet listening.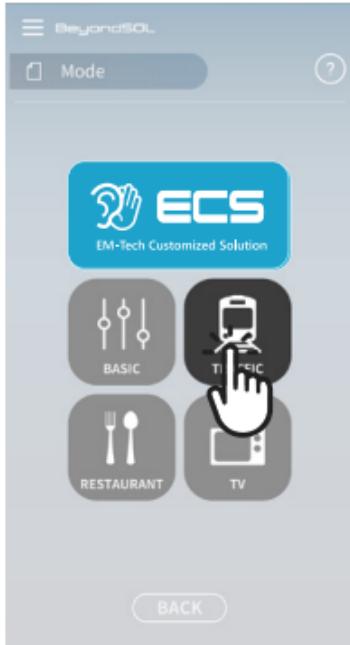


'Traffic' mode is set up to be suitable for public transits.

Application

Mode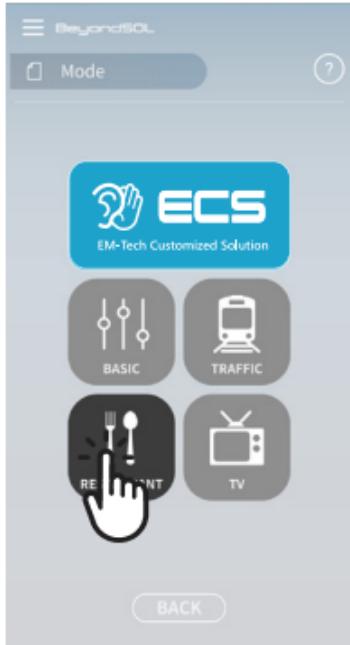

'Restaurant' mode is set up to be suitable for listening in crowded restaurants.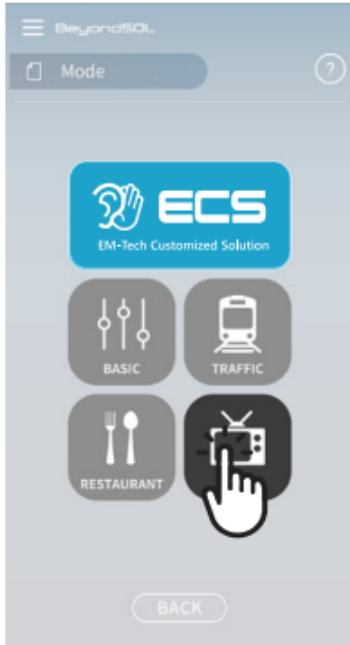

'TV' mode is set up to be suitable for watching TV.

Application


Volume Control


1. Use the volume control bar to adjust the overall gain. Adjust volume & treble / bass comfortably after the mode has changed. It can be saved for 4 individual modes.
2. Using the Bass(Low frequency), Treble(High frequency) control bar, you can create the most comfortable listening environment.
 - * If you feel uncomfortable with 'howling' or 'ringing' noise, reduce the Bass.
 - * If you feel uncomfortable with 'bowl clashing' or 'paper flipping', reduce the Treble.

Application

Preference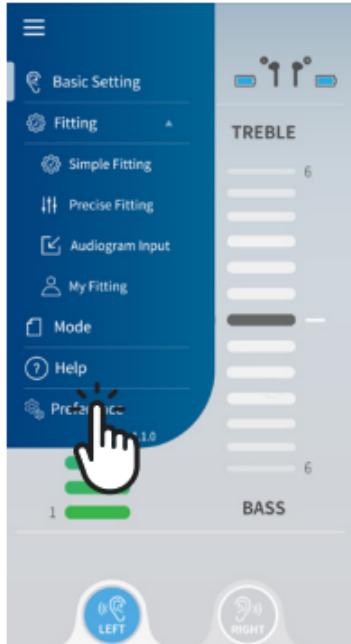

Select menu (≡) to open the slide menu and select 'Preferences'

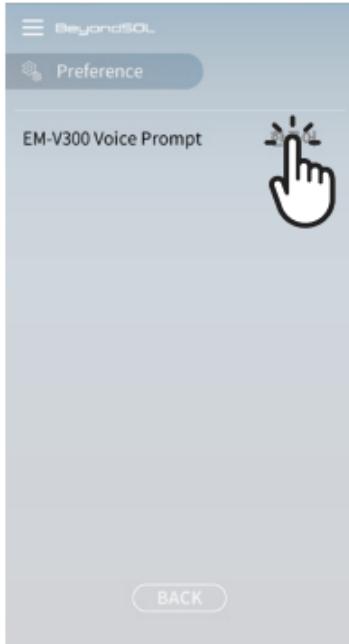The voice prompt currently set for the product is displayed on the right, and if you select it, you can change the voice prompt of the product.

Application

Preference

Select the preference language and
it will be applied to the product.

Product Warranty

- Warranty Period (Free of Charge/cost of charge service) is 1 year.
- Rather if it free of cost of charge, every A/S product including ear tip, and others will be at personal expense when mailing the product
- Product returns and refunds will be processed to the following rules.
 - Within 14 days after the purchase of the product, replacement or refund for any problems caused by defect in the product.
 - If the purchase receipt can not be checked, the warranty period of the product is calculated from the date of manufacture.
 - we will repair free of charge the trouble that occurs during normal use within the warranty period.
- Even under the warranty period, repair cost may occur in the following cases.
 - Failure or damage due to user's carelessness or misuse
 - Failure or damage caused by user's repair modification, disassembly, cleaning, etc.
 - Failure or damage due to natural disaster such as fire, earthquake, flood, lightning
 - Failure or damage due to improper storage such as falling, impact, pressure, dust, mud, sand
 - Failure or damage due to carelessness in storage (hot, humid place, cold)
 - Failure or damage caused by consumables or products other than normal products
 - If the validation is not possible (eg. if the serial number of the product is damaged)

Please check with the place of purchase or the following service center when a product breakdown occurs.

Repair Inquiry

+82-33-243-8384

Visit Inquiry

Suite 104 GangwonTechnoPark ChuncheonVenture 2nd Factory,
Sinbuk-ro 61-14, Shinbuk-eup, Chuncheon-si, Gangwon-do, Korea

Model	EM-V300		
Date of Purchase	Year	Month	Day
Place of purchase and Tel No.			
Manufacture No.			

We will notify you of the compensation within 7 days from the request date, and the compensation will be resolved within 14 days from the notification date.

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